

140mm

190mm

Instructions for Use



Product Name: Particulate Respirators

Model: N9501F

Intended Use:

The Particulate Respirator is intended to cover the nose and mouth of the wearer to help reduce wearer exposure to airborne particulates and protects the wearer from splash and spray of body fluid.

Warning:


1. Select the respirator type and determine the suitability of the respirator according to the intended use.
2. The respirator shall be abandoned when it is broken, becomes difficult to breathe, or polluted by body fluids.
3. This respirator is disposable, do not reuse or use for more than one shift.
4. Check the respirator before use. Make sure there are no broken earloops, no holes or other damages on the respirator. If any damage detected, please do not use and replace with a new one immediately.
5. The effectiveness will be reduced if facial hair or beard passes under the face seal.
6. The respirator should only be used in intended environment. Failure to follow the instruction may cause harm to health or reduce the effectiveness.
7. The respirator cannot provide intended protection if the mask is not correctly donned or is damaged.
8. Never alter or modify this respirator in any way.
9. This respirator does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life.
10. The respirator has a shelf-life of 3 years. After the expiry date the respirator must no longer be used.

Storage:

The product should be stored in an environment with suitable temperature and ventilation, protection from direct light and moisture.

Limitations of Use: None

Application Instruction:

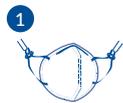

Take out the respirator and pull on the adjustable buckle with wire or other small gadget



Place respirator over face with the nose clamp sitting upward as shown. Position both Earloops behind your ears, making sure your lower chin is covered.



Hold the nose clamp to the shape of your nose to ensure it fits your face comfortably.



Adjust the respirator with adjustable buckle to a proper position and ensure there is no leakage.



To test fit: cup both hands over the respirator and inhale vigorously, a negative pressure should be felt inside the respirator. If air leaks at the edge, reposition the respirator for better fit.

This product has been certified to PPE Regulation (EU) 2016/425 according to Technical Specification to Annex II: BSI's PPE for Healthcare Professionals 2020/403 – RPE Technical Specification.

Notified Body: BSI Group The Netherlands B.V.

Address: Say Building, John M. Keynesplein 9, 1066 EP Amsterdam | P.O. Box 74103, 1070 BC Amsterdam The Netherlands

Manufacturer:

Zhende Medical Co., Ltd.

Gaobu Town, 312035, Shaoxing, Zhejiang, China



Keep Dry



Away from direct sunshine



Single Use



2797

Note: This respirator is manufactured for COVID-protection only. This filtering half mask is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.

Version: 03
Rev. Date: 2020.07.07

287U